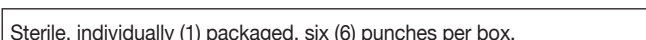
CleanCut® PerfectCut®	Rotating Aortic Punch
	Rotating Aortic Punch
Sterile, individually (1) packaged, six (6) punches per box.	

INDICATIONS FOR USE:

Aortic punches are intended to create an opening(s) in the wall of the aorta or other selected vessels to prepare a site for anastomosis. Discard after single-patient use.

PRECAUTION:

Stirle unless package is open and damaged. Before use, inspect the device to verify no damage has occurred during shipping and handling.

INSTRUCTIONS FOR USE:

- Activate the punch several times before surgical use.
- Apply a deep side-biting vascular clamp or other appropriate instrument to the aorta and make an incision in the occluded portion of the clamped vessel. This incision should be approximately 1 mm smaller than the punch tip diameter.
- CleanCut: Position the punch so the avil aligns with the incision and insert into the aorta. Then position the punch perpendicular to the aortic wall.
PerfectCut: Hold the punch perpendicular to the aortic wall.Center and insert the punch tip in the incision.
- Push the plunger towards the handle in the manner of a hypodermic syringe. There should be a perceptible snap as the die cylinder passes the punch tip, producing a clean, circular hole.


WARNING: Remove the punch from the aorta without releasing the plunger to ensure removal of the excised plug of the aortic tissue. It is possible to make any number of openings for an aortic bypass procedure with one CleanCut®/PerfectCut® Aortic Punch. Once the punch is removed from the aorta, release the handle and remove excised plug from the punch before using it in an additional procedure.

Note: After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal regulations.

DISCLAIMER OF WARRANTIES:

WARNING: Despite the exercise of all due care in design, component selection, manufacture and testing prior to sale, this CleanCut®/PerfectCut® Aortic Punch (the Aortic Punch) may fail to function due to a variety of causes including but not limited to: medical complications and stting of the CleanCut®/PerfectCut® Aortic Punch during use. In addition, Aortic Punches may be easily damaged before or during use by improper handling or other intervening acts. Consequently, no representation or warranty is made that failure of Aortic Punches will not occur or that medical complications will not follow.

DISCLAIMER: Although this Aortic Punch has been manufactured under carefully controlled conditions, manufacturer has no control over the conditions under which this Aortic Punch is used. Manufacturer, therefore, provides the following disclaimer: Aortic Punches are sold in as is condition. Subject to any express written warranties, limitations or disclaimers by manufacturer to any party, or otherwise available under law through such party, manufacturer disclaims all warranties, both expressed and implied, with respect to Aortic Punches, including, but not limited to, any implied warranty of merchantability or fitness for a particular purpose. Manufacturer shall not be liable to any person or entity for any medical expenses or, in direct, indirect, special, incidental or consequential damages caused by any defect, failure or malfunction of any Aortic Punch, whether a claim for such damages is based upon warranty, contract, tort or otherwise. No person has any authority to bind manufacturer to any representation or warranty with respect to Aortic Punches.

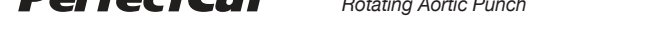CleanCut® PerfectCut®	Rotating Aortic Punch
	Rotating Aortic Punch

Long Models (8 inches) Modelles longs de 20,32 cm Modelli con lunghezza 20,32 cm Modelos largos de 20,32 cm 20,32 cm - Ausfñhrungen Modelos Complos 8 polegadas	
RCL28 RCL35 RCL40 RCL45 RCL48 RCL50 RCB40 RCB45 RCB48 RCB50	

Standard Models (6.75 inches) Modeles standard 17,14 cm Modelli con standard 17,14 cm Modelos estandar de 17,14 cm 17,14 cm - Ausfñhrungen Modelos Curtos 6,75 polegadas	
RCC28 RCC35 RCC40 RCC45 RCC48 RCC50	

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U.S. Pat. # 5,827,316 32170

For symbols glossary visit www.questmedical.com/resources.aspx	
	
One Allentown Parkway Allen, Texas 75002 USA +1.972.390.9800 +1.800.627.0226 F +1.972.390.8465	EMERGO EUROPE Westervoorsteijk 60 6827 AT, Arnhem The Netherlands
Made in USA 903501 REV L APR 23	

©2023 Quest Medical, Inc. All rights reserved.
CleanCut and PerfectCut are registered trademarks of Quest Medical, Inc.

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