Other MPS® Accessories Available from Quest Medical, Inc.

REF

- 5001104 : Arrest agent cassette
- 5001105 : Additive cassette
- 5001106 : 3/16" ID x 10' extension line
- 5001108 : Antegrade/retrograde Y-set
- 5051101 : Retrograde extension pressure line with transducer
- 5051102 : Retrograde extension pressure line with transducer and Y-set
- 5051103 : Pressure line with transducer
- 5051107 : Dual lumen retrograde extension line with Y-set

All accessories packed in case quantities of 10


One Allentown Parkway, Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465
custserv@questmedical.com | www.questmedical.com

Myocardial Protection System Delivery Sets

REF	Blood inlet inner diameter	Blood inlet length	Extension line length	Arrest agent and additive cassette included
5001100	(6.35 mm)	42 in. (106 cm)	120 in. (305 cm)	
5001102	(6.35 mm)	72 in. (183 cm)	120 in. (305 cm)	√
5001102-AS	(6.35 mm)	72 in. (183 cm)	72 in. (183 cm)	√
5001103	(6.35 mm)	42 in. (106 cm)	72 in. (183 cm)	√
7001101	(4.8 mm) plus adapter	72 in. (183 cm)	Sold separately	√
7001102	(6.35 mm)	72 in. (183 cm)	Sold separately	√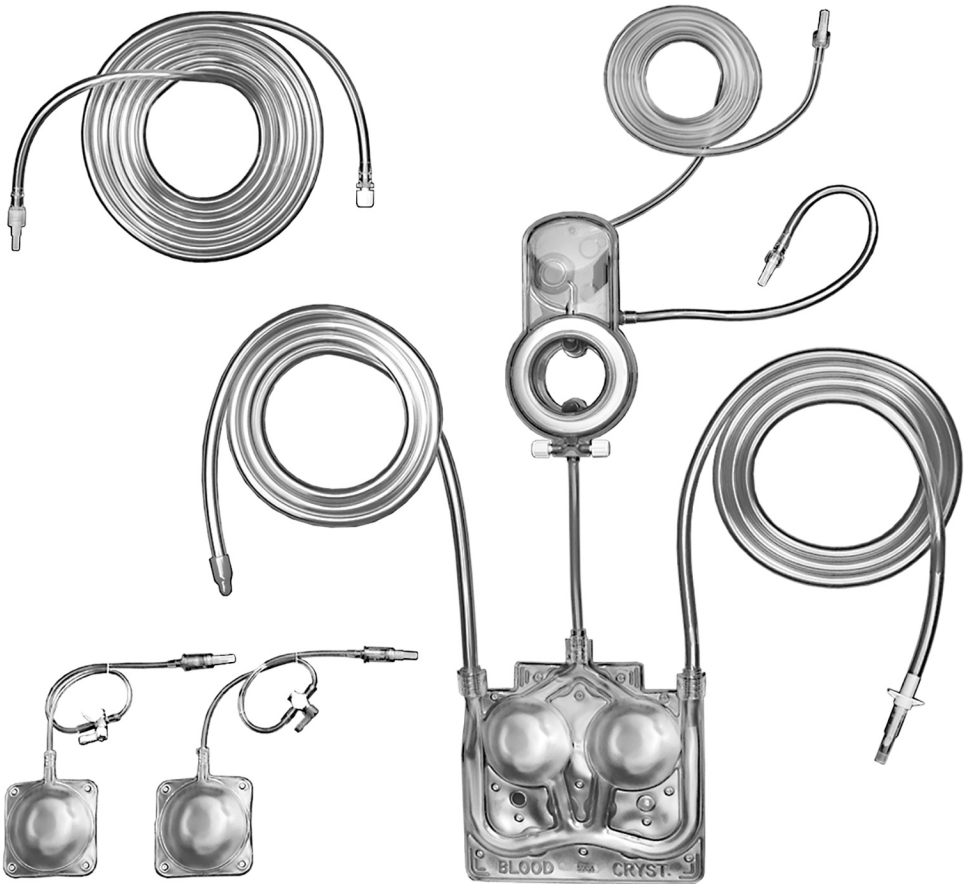

For symbols glossary visit www.questmedical.com/resources.aspx

One Allentown Parkway, Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

© 2024 Quest Medical, Inc. MPS is a registered trademark of Quest Medical, Inc.

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands

Made in USA
903962 REV H
JAN 24

MPS® Delivery Sets
Instructions for Use

Description

The Main Delivery set contains the following interconnected parts:

1. Crystalloid Line
2. Heat exchanger with integral bubble trap
3. Vent Line

REF	Blood inlet inner diameter	Blood inlet length	Extension line length	Arrest agent and additive cassette included
5001100	(6.35 mm)	42 in. (106 cm)	120 in. (305 cm)	
5001102	(6.35 mm)	72 in. (183 cm)	120 in. (305 cm)	√
5001102-AS	(6.35 mm)	72 in. (183 cm)	72 in. (183 cm)	√
5001103	(6.35 mm)	42 in. (106 cm)	72 in. (183 cm)	√
7001101	(4.8 mm) plus adapter	72 in. (183 cm)	Sold separately	√
7001102	(6.35 mm)	72 in. (183 cm)	Sold separately	√

Indication For Use

The MPS extension lines and delivery sets are only for use with the Quest MPS and MPS2 consoles for delivery of cardioplegia solutions to the heart during cardiopulmonary bypass surgery.

Contraindication

This device is designed for use with only the Quest MPS and MPS 2 console. Use with other systems is contraindicated. This device in not designed, sold or intended for use except as indicated.

Warnings And Cautions

1. Read and understand the information in these instructions prior to operating the MPS console. See Operations Manual for a complete list of Warnings and Cautions and Storage Conditions. The attending clinician is solely responsible for the setup and use of the MPS console/perfusion system.
2. Examine ALL sterile packages carefully before opening to confirm the pack-ages' integrity and verify that the expiration date has not passed. The devices are supplied in a sterile, single use package and are non-pyrogenic. DO NOT USE a damaged or opened package or if the expiration date has passed. Do not resterilize or reprocess.
3. Observe aseptic technique with all tubing connections. Do not over-tighten rigid connections. Use proper sterile technique when passing lines into the sterile field.
4. Dispose of this device according to hospital procedure for contaminated material.
5. Observe all additional warnings and cautions contained in the following Instruc-tions For Use. The main door must be closed to operate the MPS console.
6. Do not use force to shut the doors. Forcing a door closed may indicate an improperly installed or overfilled cassette.
7. Do not prime the delivery set more than 12 hours prior to surgery Biological contamination is a risk after more than 12 hours.

Installing The Heat Exchanger

1. Turn the MPS console power switch off and then on to ensure the pump pistons are retracted. Open the red inner door to reveal the pumping chambers.

 DO NOT LOAD cassettes if the pistons are in the forward position (convex). Cycle the power to reset the pistons. If the pistons do not retract. STOP THE MPS CONSOLE and call for service. If the pump piston is not fully retracted, there may be an unintentional bolus delivery during the onset of flow.

2. Verify the heat exchanger latch is in the unlocked position (rotate CCW).
3. Using aseptic technique, open the packaging of the MPS delivery and set the extension delivery line package aside.
4. With one hand, grasp the top of the heat exchanger bubble trap with delivery set tubing and remove the entire set except the arrest agent cassette from the tray.
5. To ensure proper installation when installing the heat exchanger, allow the blood:crystalloid cassette to lay inside the red inner door and verify that the blue locator ball is on the right side of the cassette. The locator ball ensures that the blood and crystalloid lines are positioned correctly. The red inner door will not close if the ball is on the left side.
6. While allowing the blood: crystalloid cassette to lay inside the red door, lift the heat exchanger vent line and position the heat exchanger on the mounting block.
7. Applying a constant pressure, seat the heat exchanger on the heat exchanger mount block. Turn the latch clockwise to lock and seal the heat exchanger on the MPS console and automatically engage and connect the water ports and the pressure, level and temperature sensors.
8. Route the vent line through the vent valve on the back of the heat exchanger. To open the valve, press down and hold open while inserting line.

Installing Blood: Crystalloid Cassette

1. Secure the blood: crystalloid cassette by firmly seating cassette over the 13 mounting pins. Carefully close and latch the red inner door.
2. Route the blood and crystalloid source lines through the mounting grooves on the left and right side of the console. Ensure that the blood and crystalloid lines do not twist and occlude the cassette inlet channels.

 Avoid OPERATOR INJURY, make sure that all fingers are clear from the doors' edges before closing the inner and outer.

Close the outer door before using the MPS console. The outer door must be closed to operate the MPS console. **HOWEV-ER, do not use force** to open or close the doors. Forcing a door closed may indicate an improperly installed or overfilled cassette.

Connecting MPS Delivery Set To Extracorporeal Circuit

If the Mini-Prime set, 7001101, is to be connected to a 1/4 in. (6.35 mm) cardio-plegia source line, then the tubing adapter (item 11) in the package must be installed as given in Step 1 below:

1. MINI-PRIME:
 - Remove the separately packaged tubing adapter from the MPS tray. Use sterile technique to assemble. Uncoil the blood line. The length can be adjusted by cutting it to desired length. Priming volume is reduced by 5 ml per foot of tubing removed.
 - Push the blood line over the tubing adapter's barb. Sterile alcohol wipes may be used to assist the assembly. During priming, ensure that the adapter connection is not leaking before patient use.

 During priming, ensure that the adapter connection is not leaking before patient use.

After the adapter is attached, proceed to Step 2 below:

2. All MPS Delivery sets:
 - Connect the delivery set / blood line to the cardioplegia outlet on the blood oxygenator.
 - Uncoil the delivery set crystalloid line and clamp the line near the cassette.
 - If necessary, insert the spike into the crystalloid bag and hang crystalloid bag. The crystalloid bag must hang at least 3 feet (1 meter) above the pumping chamber to ensure adequate pressure to fill the pump
-  The crystalloid bag must hang at least 3 feet (1 meter) above the pumping chamber to ensure adequate pressure to fill the pump.
- Route the cardioplegia delivery line through the air in line detector and antegrade delivery valve (A). To open the valve, press down and hold while inserting the line. FULLY INSERT the tubing through the air in the line detector for proper MPS console operation.
-  FULLY INSERT the tubing through the air in the line detector for proper MPS console operation.
- Uncoil the vent line and attach the luer connector to the non-pressurized luer port of the cardiotomy reservoir. Route the vent line through the vent valve. To open the valve, press down and hold while inserting the line.
- Following catheterization, connect the extension line to the delivery catheter.

Installing And Filling The MPS Arrest Agent And Additive Cassettes

 **DO NOT OVERFILL OR OVERPRESSURIZE** arrest agent or additive cassettes. The normal maximum cassette capacity is 50 ml. Overfilling the arrest agent or additive cassettes before or after installation in the console can cause unintentional bolus xdelivery during the onset of flow and possible system malfunc-tion. If the MPS console senses that the cassette contains 50 ml or more of additive agent, the “Additive (or ARREST) IS OVER-FULL Remove some fluid/CONTINUE” screen displays. Do not ignore this message and overfill the cassette.

1. Open the yellow and green doors, of the console to expose the pump chambers.
2. Secure the arrest agent cassette (yellow stopcock) over the mounting pins behind the yellow door and route the infusion line up through the channel directly above the pump chamber.
3. Secure the additive cassette (green stopcock) over the mounting pins behind the green door and route the infusion line up through the channel directly above the pump chamber.
4. Carefully close the yellow and green inner doors.
5. Remove the additive Luer CAP PORT from the heat exchanger and connect the normally closed checkvalve.
6. Remove the arrest agent Luer cap from the heat exchanger and connect the normally closed checkvalve.
7. Fill a syringe with 50ml of **ARREST AGENT** solution with a concentration of 2mEq/ml.
8. Connect the filled syringe to the yellow stopcock and turn the handle to the fill position.
9. Hold the filled syringe in an upright position and withdraw the plunger to ensure the cassette is free of air. Carefully fill the cassette by pressing the plunger.
10. Disconnect the syringe from the Luer port on the stopcock and turn the stopcock to the delivery position.
11. Fill a syringe with no more than 50 ml of ADDITIVE solution mixed to the proper concentration.
12. Connect the filled syringe to the green stopcock and turn the handle to the fill position.
13. Hold the syringe in an upright position and withdraw the plunger to ensure the cassette is free of air. Carefully fill the cassette by pressing the plunger.
14. Disconnect the syringe from the Luer port on the stopcock and turn the stopcock to the delivery position. During the Console setup procedure, set the desired additive delivery concentration between 0 and 50 ml/L of cardioplegia solution. During the console setup procedure, set the desired high and low arrest agent delivery con-centration between 0 and 40 mEq/L of cardioplegia solution.

 During the Console setup procedure, set the desired additive delivery concentration between 0 and 50 ml/L of cardioplegia solution.

During the console setup procedure, set the desired high and low arrest agent delivery concentration between 0 and 40 mEq/L of cardioplegia solution.

Priming The System And Setup

Case parameters must be set before the system can be primed. Prior to delivering cardioplegia, the water circulation system is purged in parallel with the automated delivery set priming to remove all air in the system. The MPS unit must successfully complete the purge and prime prior to cardioplegia delivery. Refer to the MPS console Operators Manual for purge setup and operating instructions.

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