**REF** 530NA16 MPS<sup>®</sup>3 Power Cord  
530CE16 MPS<sup>®</sup>3 Power Cord  
530IT16 Cavo di alimentazione MPS<sup>®</sup>3  
530SA16 MPS<sup>®</sup>3 Power Cord

#### DESCRIPTION:

530NA16 - 16 ft. length power cord with hospital grade NEMA5-15 plug and IEC320 C13 connection.

530CE16 - 15 ft. length power cord with hospital grade EU1-16P plug and IEC320 C13 connection.

530SA16 - 8.2 ft. length power cord with hospital grade UK2-15P plug and IEC60320 C13 connection.

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#### INDICATION FOR USE

The MPS<sup>®</sup>3 Power Cord is indicated to be used to connect the MPS 3 console to a hospital-grade electrical outlet.

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#### CONTRAINDICATION

This device is designed for use with the Quest MPS 3 console. Use with other systems is contraindicated. This device is not designed, sold, or intended for use except as indicated.

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#### WARNINGS

- Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.
- For use with Quest's MPS3 console only. All claims void if used separately.
- The hospital-grade electrical outlet or appropriate hospital outlet should have proper grounding.

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#### CAUTIONS:

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- Observe all additional warnings and cautions contained in the following Instructions For Use.
- The MPS 3 console must be plugged into a hospital-grade electrical outlet or appropriate hospital outlet on an isolated circuit from the heart-lung machine.

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#### INSTRUCTIONS FOR USE

1. Install C13 plug end of cord into the MPS 3 power receptacle (on rear of unit) with cord facing toward the back of the unit.
2. Plug other end of power cord into a hospital-grade electrical outlet or appropriate hospital outlet.

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