


QUEST MPS® 3

Smarter Myocardial Protection™

MPS® 3 Drug Pump Cover

REF 5303031

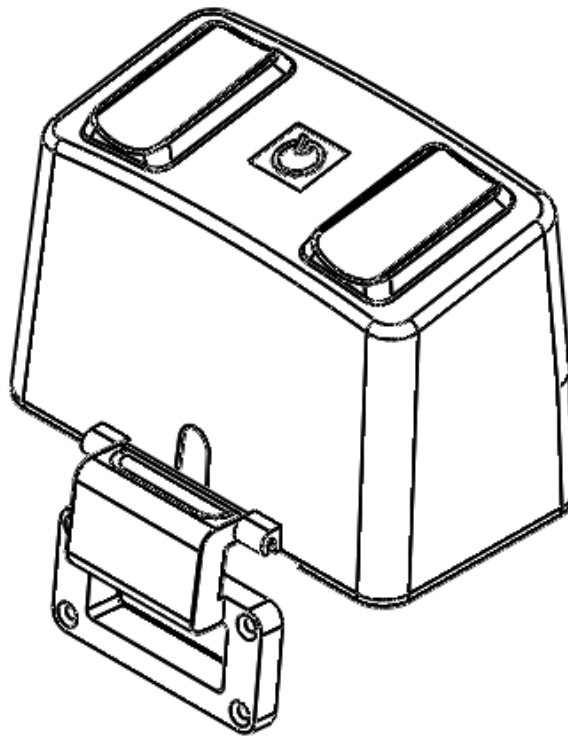

For symbols glossary visit www.questmedical.com/resources.aspx



One Allentown Parkway, Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands

© 2023 Quest Medical, Inc. MPS is a registered trademark of Quest Medical, Inc. All rights reserved. The MPS Console and MPS sterile disposables are covered under one or more of the following U.S. patents: 5,385,540 . 5,573,502 . 5,588,816 . 5,638,737 . 5,645,531 . 5,899,873. Also covered by pending U.S. and International Patents and patent applications.

Made in USA
904679 REV A
MAR 23

QUEST MPS® 3

Smarter Myocardial Protection™

REF 5303031 **MPS® 3 Drug Pump Cover**

DESCRIPTION

MPS® 3 Drug Pump Cover

INTENDED USE

The MPS® 3 Drug Pump Cover is indicated to be used with the Quest MPS® 3 console to deter fluid intrusion into the console.


CONTRAINDICATIONS

This product is designed and warranted for use with the Quest MPS® 3 cardioplegia delivery system only. Use with other systems is contraindicated. This product is not designed, sold, or intended for use except as indicated.


WARNINGS AND CAUTIONS

1. Read instructions carefully.
2. For use with the Quest MPS® 3 System only. All claims void if used separately.

UNINSTALLING THE MPS® 3 HANDLE

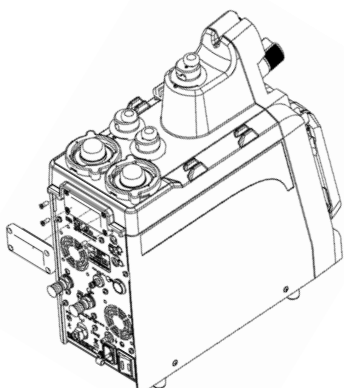


1. Using a Philips head screw driver, carefully remove the (4) screws on the MPS handle currently attached to the MPS Unit.
2. Place (4) screws aside to use later.

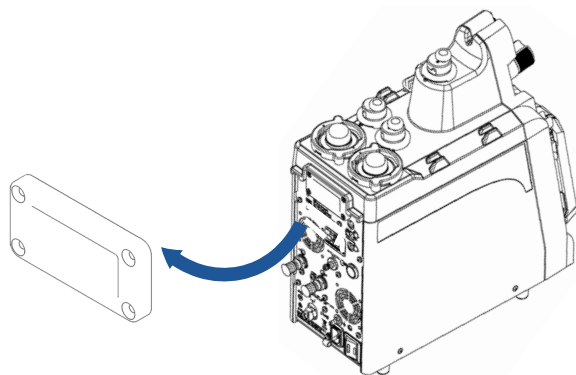

3. Pull on handle to remove from the MPS Unit.


INSTALLING THE MPS® 3 DRUG PUMP COVER

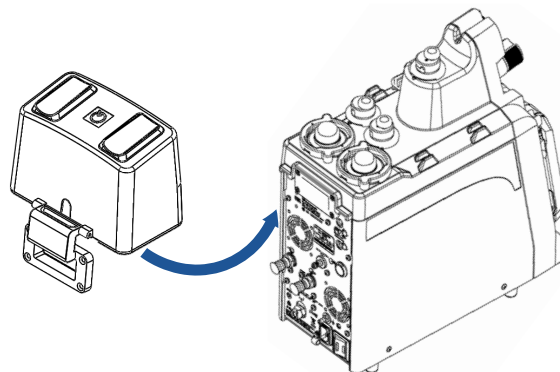

1. Take the Drug Pump Cover and fit handle into MPS Unit. Push handle portion of pump cover into the MPS Unit.
NOTE: The fit is snug, You should hear the handle rub or snap into place. This sound is indication of a good fit.
2. Once Drug Pump Cover and handle are in place secure with (4) screws using a Philips screw driver. Tighten to snug.

INSTALLING THE MPS® 3 DISPOSABLE SET

1. First follow section 8.1.1, in the Operations Manual for heat exchanger installation.

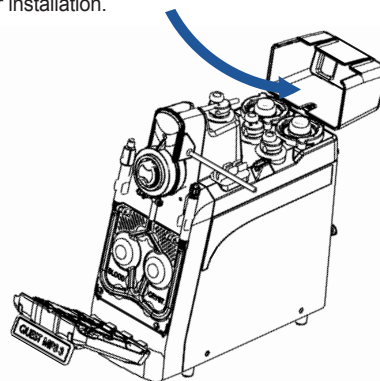

2. Tilt the Drug Pump Cover open and advance the vent line through the back opening of cover. Once line is fed through the back opening, follow steps for remaining portion of section 8 in the Operations Manual for disposable installation.