

REF 5303011



DESCRIPTION

MPS 3 Ohmeda Medical Air Adapter Kit. Use this kit to connect Ohmeda Medical Wall Outlets to the MPS 3 Medical Air Hoses.

INDICATION FOR USE

MPS 3 Ohmeda Medical Air Adapter Kit is indicated to use as the interface between the MPS 3 Medical Air Hose and the hospital's installed medical air system.

CONTRAINDICATION

This product is designed and warranted for use with the Quest MPS 3 cardioplegia delivery system only. Use with other systems is contraindicated. This product is not designed, sold, or intended for use except as indicated.

WARNINGS and CAUTIONS

1. Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.
2. Federal (U.S.A) Law restricts this device to sale or on the order of a physician.
3. Do not kink tubing.
4. Before connecting Medical Air Supply to the MPS 3 System, please ensure the maximum pressure does not exceed 55psi. While the MPS 3 System will regulate the pressure down to the necessary operating levels, connecting high pressure may damage internal components.
5. Avoid OPERATOR INJURY. Only qualified technicians should attempt to connect a Medical Air Supply to the MPS 3 System. Make sure that all fingers are free from pinch/crush hazards while connecting the Medical Air Supply and tubing connectors.
6. Please ensure that the Medical Air Supply is free from oil, water, and residue to prevent unintended damage to the MPS 3 System.

INSTRUCTIONS FOR USE

1. Plug the brass and black quick connect of the Medical Air Hose into the back of the MPS 3 Console by pushing it into the mating quick connect (See Figure 1 and 2).
 - a. If needed, refer to the Medical Air Hose IFU and MPS 3/ND User Manual.
2. Plug the yellow end of the Medical Air Hose into the Ohmeda Adapter (See Figure 2 and 3).
3. Plug the Ohmeda Adapter into Ohmeda Wye Block by aligning the rectangular keys with the indented features on the wye block.
4. Plug the Ohmeda Wye block into the medical air wall outlet (See Figure 4)
5. To disconnect Ohmeda connections grab the yellow feature of the Ohmeda male plugs and rotate clockwise to compress the internal locking spring. Once locking mechanism is disengaged pull backwards on the male Ohmeda Male plug to disengage the Ohmeda quick connect (See Figure 4).

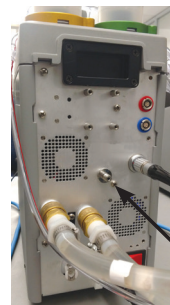


FIGURE 1

Brass and black quick connect to connect to the MPS 3

Yellow end to connect to the medical air wall outlet

MPS 3 medical air inlet

FIGURE 2

End that connects to the medical air wall outlet

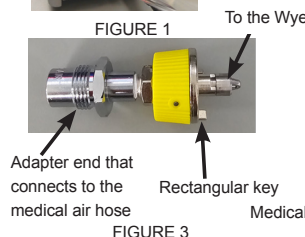


FIGURE 3

To the Wye

Adapter end that connects to the medical air hose

Rectangular key

Medical Air Hose



FIGURE 4

End that connects to the Ohmeda Adapter

For symbols glossary visit www.questmedical.com/resources.aspx



Rx ONLY



QuestMedical.com/eIFU



One Allentown Parkway, Allen, Texas 75002 USA

+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

© 2023 Quest Medical, Inc. MPS is a registered trademark of Quest Medical, Inc. All rights reserved. The MPS Console and MPS sterile disposables are covered under one or more of the following U.S. patents: 5,385,540 . 5,573,502 . 5,588,816 . 5,638,737 . 5,645,531 . 5,899,873. Also covered by pending U.S. and International Patents and patent applications.

Made in USA

904705 REV A

OCT 23