

Additional information concerning MPS® 3 and MPS® 3 ND

Dear Quest Customer,

MPS 3 and MPS 3 ND SW has been updated to Rev 10.3.

Edits have been made to the Error Code List in the Troubleshooting section 11.3 of the user manual.

The Alarm code and description here supersedes the listing in the user manual.

Alarm Code	Message Display	Corrective Action
197	Arrest pump piston sensor error Verify Arrest cartridge installation DISABLE	* Select DISABLE to complete Prime * Check Arrest Cartridge to verify it is properly installed. * Retry drug Prime from the Home screen. * If error still persists, contact Service
201	Additive pump piston sensor error Verify Arrest cartridge installation DISABLE	* Select DISABLE to complete Prime * Check Additive Cartridge to verify it is properly installed. * Retry drug Prime from the Home screen. * If error still persists, contact Service